



# **Medicalproducts Ltd, Inc.**

**Mail: P.O. Box 80685, Conyers GA 30013-0685**  
**Phone: (678) 413-9200 • Fax: (770) 860-8845**  
**Toll Free Orders: (800) 345-2922 • Fax: (800) 372-5649**

Dear Valued Customer,

We are writing to re-introduce our **MASK-695 NIOSH Approved Respirator** because of the ever present risk of the resurgence of Tuberculosis worldwide, risks in exposure to biohazard agents and always present virus risk when treating patients or handling deceased persons.

## **Why N-95 Particulate Respirators?**

Federal law (OSHA 29 CFR Part 1910.134) requires **all healthcare personnel** who may come in contact with possible TB-positive patients have access to a NIOSH approved Particulate Respirator. **These respirators differ from standard surgical masks in that they must be fit-tested to afford a proper facial seal.**

## **Features and Benefits**

### **One-Size-Fits-All**

While competing products necessitate ordering and stocking various sizes, the 695 Respirator is a true **"One-Size-Fits-All"**

### **Positive Facial Lock**

Unique locking mechanism insures a custom fit to any size face.

### **Traditional Pleated Design**

Standard **Professional Look**  
You don't look conspicuous or "funny".

### **Fluid Resistant**

Safety; the 695 mask passes all industry standards for fluid resistance.

### **Soft, Cloth-like Bands**

Latex-free soft headbands that **don't tangle your hair.**

To request additional samples, please contact us toll free at **800-345-2922**, fax at **800-372-5649** or Email at [sales@medicalproducts.com](mailto:sales@medicalproducts.com) or thru our website at [www.medicalproducts.com](http://www.medicalproducts.com).

We want to take this opportunity to thank you for your continued support of us at Medicalproducts

Best regards,

George "Andy" Everson  
Sales Manager

## MASK-695 Respirator Donning Procedures \*



### Step 1

Grasp nosepiece at the top of mask with one hand. Slide finger of other hand behind chin piece.



### Step 2

Open mask by pulling down and out on chin piece until mask is fully open. Place chin in pocket created. Stretch bands over back of head, positioning bottom band behind neck below ears and top band near crown of head.



### Step 3

Conform nosepiece to nose and face contours making sure Magic Arch<sup>®</sup> is fully extended providing maximum clearance from nose and mouth.



### Step 4

Holding mask at nosepiece, place one finger inside the mask on the malleable aluminum chin piece. Pull down gently so that mask is snug against the face.



### Step 5

Pinch any excess material together under chin and fold over or twist.



### N95 Respirator Correctly Fit to Face

**FACE FIT CHECK** – To check fit, place both hands along edges of the respirator and exhale. If air escapes around your nose, adjust nosepiece. If air leaks at the respirator edges, unfold or untwist the chin piece and repeat step 4 and 5 above.

**NOTE:** This procedure should be done the first time in conjunction with an OSHA Approved Fit Test Kit to be in "compliance" with OSHA Respiratory Standard 29 CFR 1910.134 and manufacturers recommendations. Contact us for a copy or go to [www.osha.gov](http://www.osha.gov) website or [www.medicalproducts.com](http://www.medicalproducts.com)

\* Instructions were provided to us by the manufacturer and provided herein to you for your convenience by:



**Medicalproducts Ltd., Inc.**

Toll Free (800) 345-2922 • Fax (800) 372-5649 • Ph: 678-413-9200  
Website: [www.medicalproducts.com](http://www.medicalproducts.com) Email: [sales@medicalproducts.com](mailto:sales@medicalproducts.com)